


Multi-Functional Electrotherapy Garment Wrap

Now electrotherapy can be easily applied to hard to reach spots on the body with incredible accuracy. This unique conductive mesh material can stimulate large or multiple areas of the body.

Energy Brace is designed to be used in conjunction with a wide variety of Electro-Medical modalities, both in the home and a physical therapy setting. Energy Brace comes in a variety of sizes for use on, but not limited to elbow, wrist, and ankle; knee; and back.

Energy Brace offers support, light or tight compression and stimulation simultaneously without excessive perspiration generally found with the use of neoprene supports. Not only is our mesh material more conductive than most other materials on the market, but it stretches to fit any patient and any body part.

Conductive Pads to treat pain on any body part

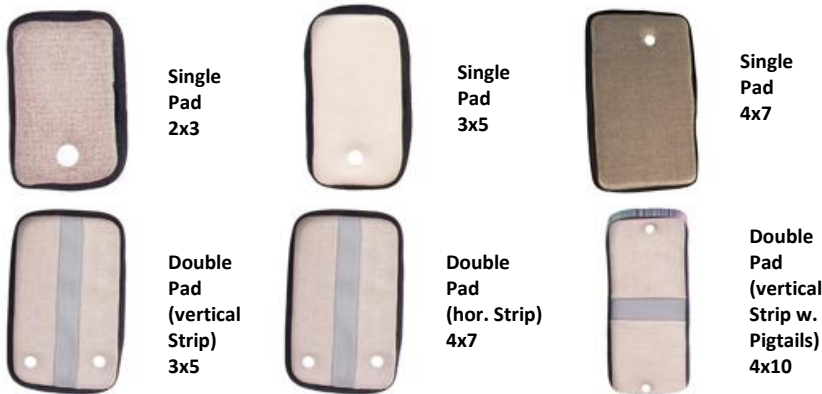

IMPORTANT

- Garments are intended for use/reuse by a single patient only.
- Hand wash in cold water using soap designed for washing delicate fabrics.
- Rinse thoroughly. Carefully block into shape on a towel and allow to air dry.
- Do not place in dryer.
- Do not apply to broken skin or open wounds.
- Garment electrodes can use either positive or negative polarity.
- No stimulation should be applied while operating a motor vehicle or machinery.



ENERGY BRACE WRAP

EB Wrap 4x40 (stretchy belt)



EB Wrap 8x40



Conductive spray, TENS unit and lead wire accessories for use with, but are not included with Energy Brace.